

尺寸: 235*110mm

材质: 300g 硬卡纸 / 半透硫酸纸

Heating Eye Mask USER MANUAL

Model: MJ01



Any quality issues, please contact us

Email: support@mannajoy.com

Website: www.mannajoy.com

Instructions for use

Power on/off:

Press and hold for 3s



Temperature switching

Press to cycle through 3 levels of temperature



MODE 1
35°C/95°F



MODE 2
45°C/113°F



MODE 3
60°C/140°F

Note: The eye mask will automatically shut down after 15 min of use.

Instructions for cleaning

After taking out the heating pad, the cover can be washed.

Note: Gentle hand wash recommended; avoid machine washing



Washable cover




Unwashable heating pad

Instructions for charging

For charging, plug the charging cable into the Type-C port under the button on the front of the eye mask.

It takes about 1 hour to fully charge.

Fully charged, it can provide 5 cycles of heating

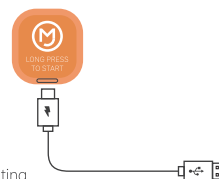

Specifications

Model	MJ01
Material	100%Satin & 100%Polyester
Dimension	11*5 inches
Mode	①35°C/95°F ②45°C/113°F ③60°C/140°F
Battery Capacity	3.7V 1200mAh

Cautions



*Do not fold while heating. Burns will result from improper use.



*Do not wash or soak the heating pad in water.



*MJ01 is not intended for diagnosing or curing any diseases, and is not a replacement for consulting. If symptoms do not improve, please consult a health care professional.

Manufacturer

Manufacturer: Changsha Beisile Technology Co., Ltd.

Address: No. 514, Building 18, Futian Yanghu Shidaiyuan, No. 177, Xueshi Road, Xueshi Street, Yuelu District, Changsha City, Hunan Province, China, 410000

Made in China